Recogemuestras con anillo de reboso
Sample insert with overflow ring

Diagnostest OVA

Sistema de diagnóstico de parásitos para dueños de mascotas
Parasite diagnostic system for pet owners

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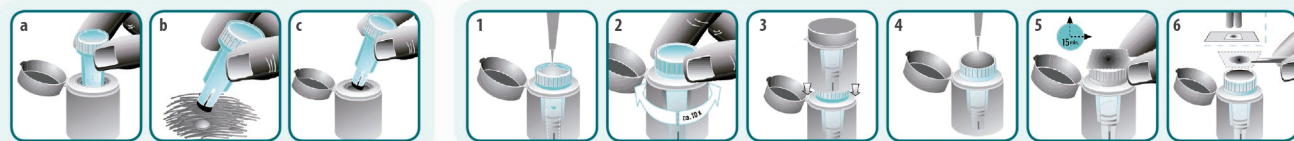
Parasite diagnostic system (flotation method) for the analysis of fecal samples of animals. Test kit for:

- A clean and easy taking of a sample by the pet owner.
- A convenient hygienic sample analysis by the veterinarian.
- Reliable accurate test results.

How works Diagnostest OVA:

Suspending the fecal sample in the flotation medium which has a specific weight of 1.2, the ova of pseudocoelomates and tapeworms as well as coccidia oocysts float to the top of the insert. The heavier parts of the sample remain on the bottom of the container. The ova or oocysts then can be taken/transferred easily to a slide for microscopic examination.

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Collecting:

- Lift the cap and remove blue insert.
- Push the small end of blue insert into the stool to collect a fecal sample.
- Place the blue insert without locking in the container. Close the cap. Take it to your veterinary surgeon or laboratory as appropriate.

Analysis:

- Lift the cap and fill the blue insert with flotation solution to the middle mark.
- Rotate the blue insert back and forth approx. ten times to separate the parasite ova from the fecal sample.
- To secure the blue insert in the outer container, press down into place by applying pressure to a second container placed on top.
- Completely fill the blue insert with additional flotation solution to form a meniscus.
- Float a 22 x 22 mm slip on the meniscus for 15 mins. By means of flotation, the parasite ova will remain on the cover slip.
- Remove the cover slip carefully and start the microscopic examination at a maximum magnification factor of 100.